

Pets cannot tell us where
they live but
microchips can



Permanent
pet
Identification



Provides
proof
of ownership



Reunites lost
pets with
their owners



supplied **through Veterinarians only**

BackHome 
BIOGLASS
Mini Transponder

Virbac

Shaping the future of animal health

Features and benefits of the backhome website and app



24/7 365 electronic support plus a friendly call centre during office hours



Secure database



Personalized dashboards



Register ANY new microchip



Barcode scanning facility



Electronic microchip search function to rehome lost pets



Update pet and pet owner details with ease




Receive important notifications



How to download the Virbac BackHome App

1. **Search** for the Virbac BackHome App and click **Install** in your app store
2. Once the App is installed click on **Open**
3. Look for the Virbac BackHome App icon on your phone screen and click on it

How to register on the Virbac BackHome App

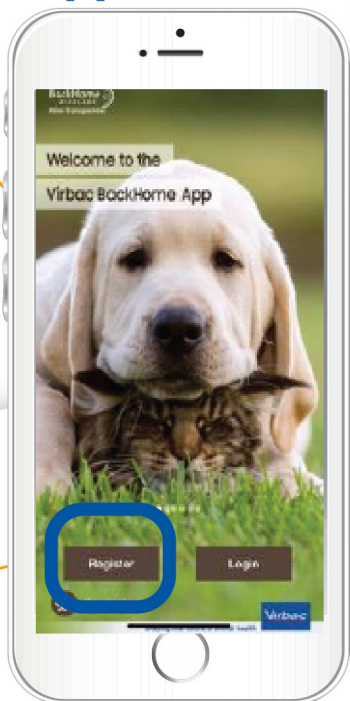

1. **Open** the Virbac BackHome App icon from your screen
2. Click on the **Register** button on any one of the screens, then click on pet owner



Reasons to insist on BackHome



- Complies with ISO 11784/ 85 standards
- Lasts a lifetime
- International company
- Microchips are manufactured according to European Standards
- No hidden costs
- Anti-migration technology
- Virbac BackHome App
- Owner can upload photos
- Control over your own profile and update your password
- Controlled change of ownership



Important facts about microchipping

- Electronic identification (“microchipping”) of pets **should only be done by veterinarians and qualified staff**
- Helps to identify the animal by a unique identification code. The procedure consists of injecting a transponder (includes a microchip, antenna and surrounding material) into an animal and that can be read by a special scanner.
- Transponders are entirely **biocompatible** and safe for the animal. The transponder is passive and is only activated when a scanner is passed over it.
- A **transponder lasts the animal’s lifetime**. The Unique number is loaded with your contact details onto the secure Backhome Database.
- **Quick and painless**. In a few animals or certain species.
- **Transponder cannot track the location of your pet.**
- Update your pet information in case of changes.
- **Ask your veterinarian** for more details.





**Get more information
and benefits on
www.backhome.co.za**

OR DOWNLOAD THE



Virbac Backhome APP



Download on the
App Store



GET IT ON
Google Play

Virbac

Shaping the future of animal health

Virbac RSA (Pty) Ltd (Reg. No. 1990/003743/07) Private Bag X115, Halfway House 1685,
Tel: 012 657 6000, Fax: 086 565 7208, za.virbac.com